SAFE AGE

WWW.SAFEAGE.EU

This product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser. This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.


EN

ASSEMBLY INSTRUCTION


STEP 1:

Put the seal upside down and attention to the Triangle marked in the above picture.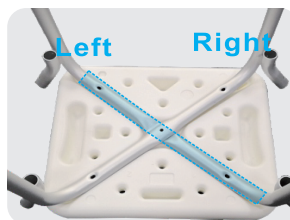

STEP 2:

Attention the Left and Right marked in the above picture, place the centrally concave down tubing on the left side and centrally concave up tubing on the right side.

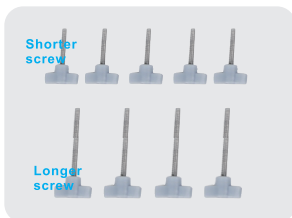STEP 3:

Please check the attached nine thumb screws like the above picture, five shorter ones and four longer ones.


STEP 4:

Insert the shorter thumb screw through the middle hole but do not fully tighten the screw.

STEP 5:

Assemble backrest by taking the "L" shaped backrest support tubings and securing them to the back rest using four shorter thumb screws like the picture above.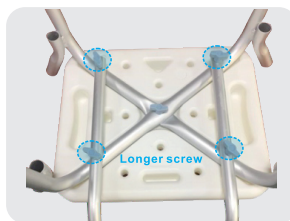

STEP 6:

Attach the backrest to the seat by aligning holes on the "L" shaped backrest tubings and fix, using four longer thumb screws like the picture above. Please fully tighten the middle screw and all other thumb screws and check to make sure all screws are fully tightened.

STEP 7:

Turn shower chair over. Slide arms into mounts and lock into place with spring loaded push-pins.

STEP 8:

Fully Assembled product. Connect chair legs and blue padding.

Periodically check assembly for continued tightness.

Never use bath safety seat if any looseness in the assembly is noticed, as personal injury may result. Sit. Do not stand on this seat to avoid dangerous fall.

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