EN

**SAFE AGE**

**ADJUSTABLE  
MEDICAL CRUTCHE**

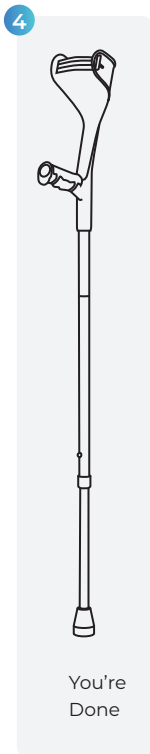
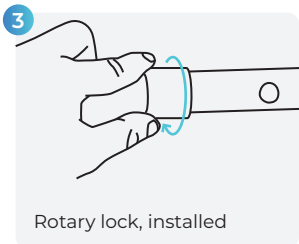
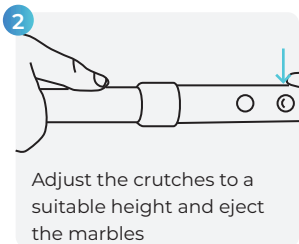
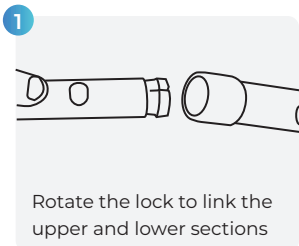
**USER MANUAL**

Please read the instructions carefully before use.

This crutch is a walking aid , please be sure to consult a physician or related medical staff before use.

Please use this product correctly to avoid accidental injuries due to improper use.

## Installation Method



**\*\*Use directly after installation and confirmation that the structure is stable.**

**It is very important that the crutch is properly adjusted. Consult a specialist if you have any doubt.**

1. Determine the height of the crutch by standing upright with your arms relaxed at your sides.
2. Adjust the crutch so that the handle is at wrist height.

### How to calculate the length of crutches:

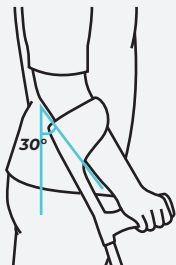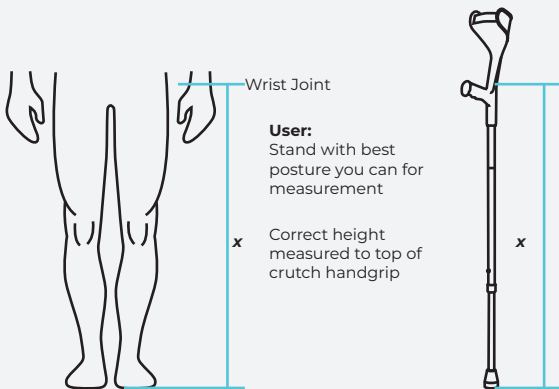(People with hunched backs just need to stand with their backs hunched)

Once you think you have the crutches at the right height you can do a quick test to check. Stand normally and take hold of the handgrips as you would if you were using the crutches. Now look at the angle of your elbow. Your elbow should be bent somewhere between 15 and 30 degrees.

## Cautions

- 1 Before use, first check whether all connectors are firmly connected.
- 2 Please keep the product away from heat sources to avoid deformation.
3. When installing the product, you must strictly follow the instructions.
4. If the product components are damaged, use is strictly prohibited.
5. Please use under supervision.
6. Please store in a dry place
7. In a ventilated environment without corrosive gas, away from fire sources and flammable materials.
8. Transport at room temperature, avoid rain and direct sunlight

## Garantie-informatie

Thank you very much for purchasing our products, so that you can better enjoy our high-quality products and services. Read the product instruction manual carefully or contact the salesperson.

Personnel understand the correct use and maintenance methods.

- A. If the damage is caused by normal use (not caused by human factors), the warranty period is from the date of purchase. Two year, lifetime service.
- B Any alteration by the customer will be invalid.
- C During the warranty period, if the product itself has quality problems and the damage is not caused by human factors, please

Return the product, and invoice/receipt to the dealer for free repair.

## Note

Please be sure to fully heed the following warnings before using this product.

Consult your dealer or health professional to avoid any physical injury or damage to your items.

Use it properly under the guidance of a doctor or therapist. If the limb pad is found to be worn and ineffective, please replace it in time.

Do not use on wet or slippery surfaces.

Make sure all fixing components are intact and secure.

Main material: aluminum alloy, rubber feet  
Maximum load-bearing 110KG  
Adjustment distance: 84-94CM

The following situations are not within the scope of free repair.

Our company has the right not to provide refunds, exchanges or free repairs.

Warranty service, but repair service can be provided (only the cost of replacement parts will be charged).

- Failure or damage caused by failure to install according to the instructions or improper use.
- The product is disassembled or repaired by unauthorized personnel of our company, resulting in failure or damage.
- Failure or damage caused by force majeure or human behavior.
- Other failures or damages caused by problems with materials or manufacturing other than the product itself.
- Consumable parts in product components are subject to normal wear and tear. Such as foot pads/foam armrests/foam grips

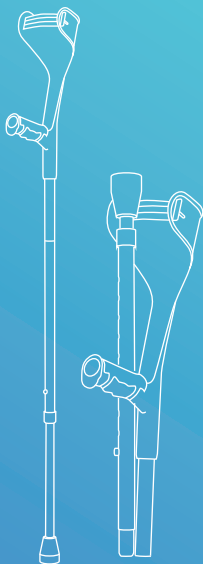
Please keep related purchase receipts properly.

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